

# VitalShield™ PRO

Provides an added layer of protection and safety, with all-day comfort and usability.

## Product Features:


- FlexFit design conforms to your head
- Ultra Lightweight
- Anti-Static clear film
- ComfortFit Pad for all day use
- Adjustable SecureFit Stretch Band




Made In the USA




## VitalShield™ PRO Assembly Instructions

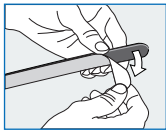


- 1) Remove items from packaging. Peel adhesive backing off foam comfort pad in preparation for applying to head band.

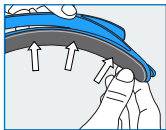


- 2) Line up foam comfort pad with inner surface of head band. Firmly press along foam comfort pad for complete attachment.

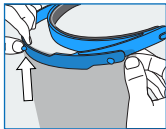

- 3) Ensure head band is positioned correctly with center feature and logo facing upwards. Place center hole of clear shield over center mounting point of head band.

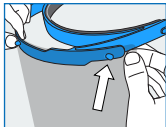

- 4) Attach outer holes of clear shield to outer attachment points of head band by pulling firmly over attachment points one at a time. Note: you may hear a clicking or popping sound as clear shield falls in place.

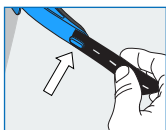

- 5) Attach one elastic comfort band (white or black) by stretching attachment holes over mounting points.



- 6) Place protective face shield over your head, adjust elastic comfort band as desired. Note: if there is an excess of elastic comfort band, overlap it back onto mounting points for seamless fit.

## Size and Fit


FlexFit headband design conforms to your head and SecureFit Stretch Band allows for adjustability - one size fits most.



**3D LOGICS**  
ADVANCED MANUFACTURING SOLUTIONS

For more information visit: [www.vitalshields.com](http://www.vitalshields.com)  
For customer support email: [customerservice@vitalshields.com](mailto:customerservice@vitalshields.com)

## What's Included

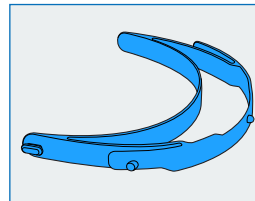


### 1x FlexFit Headband

**Cleaning Instructions:**  
Thoroughly cover all surfaces with warm soapy water, or use disinfectant cleaners and wipes. Can be steam sanitized when medically required.

**Usage:**  
Multi-Use

**Materials:**

- Polypropylene Copolymer
- Recyclable
- FDA 21 CFR 177.1520 compliant

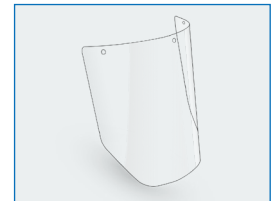


### 2x Clear Face Shield

**Cleaning Instructions:**  
Use a soft, lint-free cloth. Wipe with warm soapy water or disinfectant cleaner. Rinse with clean water and dry. Can be UV sanitized when medically required.

**Usage:**  
Multi-Use

**Materials:**

- PET
- Recyclable

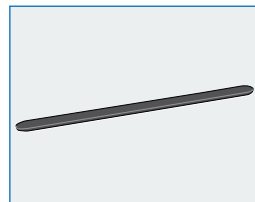


### 1x Foam ComfortFit Pad

**Cleaning Instructions:**  
Use disinfectant cleaner or wipe as needed.

**Usage:**  
Multi-Use or Single-Use (if medically required)

**Materials:**

- Polypropylene
- Acrylic Adhesive

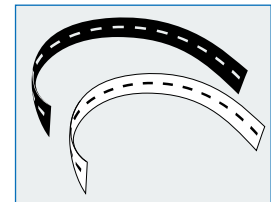

### 2x SecureFit Stretch Band (white, black)

**Cleaning Instructions:**  
Machine washable.

**Usage:**  
Multi-Use or Single-Use (if medically required)

**Materials:**

- Nylon/Polyester Blend
- Rubber Elastic